




**MEN'S
PERFORMANCE LIGHT**


QUICK START GUIDE

STEP 1

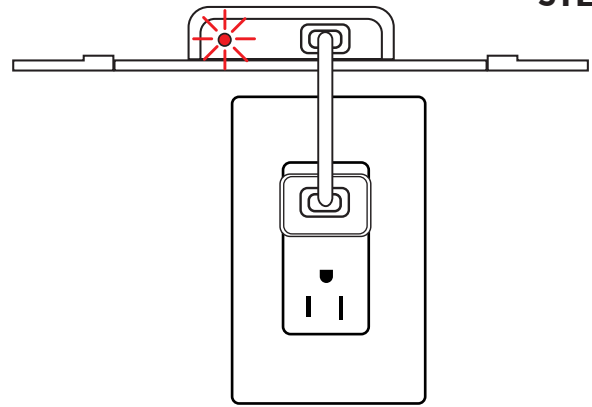


Remove the light from the box and use the included charger to ensure the light is charged prior to use. When charging, the indicator light will flash red.

STEP 2

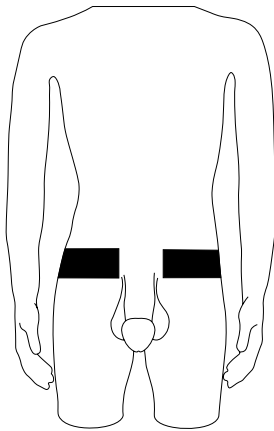


Have user disrobe, then wrap the stretchable band around their waist. The band will slide into the slots and velcro together on either side of the device.

STEP 3

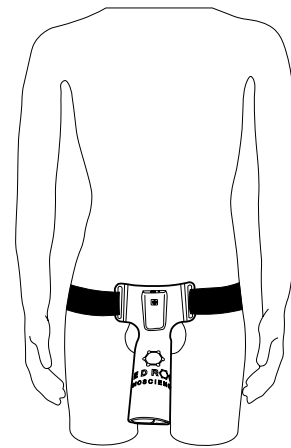


Roll the device into a hollow tube around the outside of the penis and use the magnets to secure it in place. Attach the device to the body using the stretchable waitiband by looping it around the slots on the device.

STEP 4

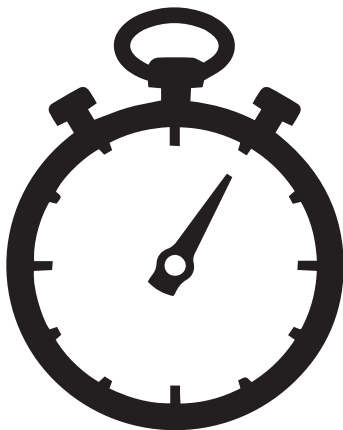

If treating for less than the standard 20 minute treatment time, get a stopwatch and start the countdown, then proceed to step 5. If treating for the full 20 minutes, proceed to step 5 now.

STEP 5

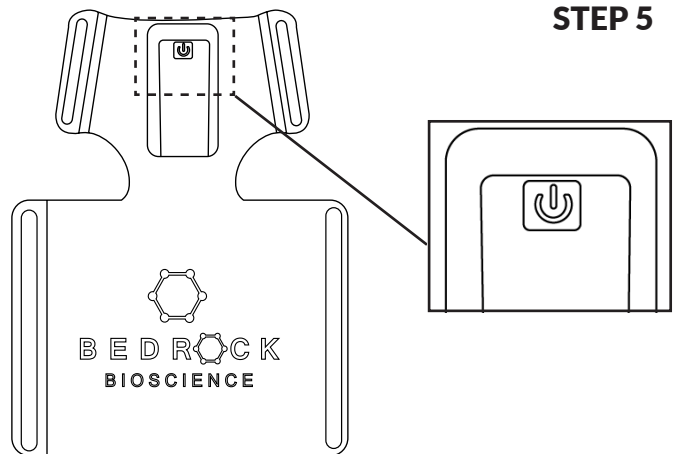

Quickly press the power button to turn the device on. The treatment will last 20 minutes and the device will turn off. If you need to stop the device early, press and hold the power button for 3 seconds to turn the device off.