



B E D R  C KTM

B I O S C I E N C E

MEN'S

PERFORMANCE LIGHT

User Guide

Includes important safety information.
Read all instructions before using the device.

The Men's Performance Light is intended for the temporary increase in local blood circulation within the treatment area. Increased circulation and blood flow have shown to enhance erectile performance.

SYSTEM INCLUDES

- Men's Performance Light
- Adjustable Strap
- USB Power Adapter
- USB Power Cord
- Storage Case
- User Guide

PRODUCT REPRESENTATION

Bedrock BioScience™ is constantly improving its products with the latest technology. As a result, the actual product may differ from the product shown in images on the packaging and in this user guide.

CONTRAINDICATIONS

- Do not use this device if your skin is light sensitive or you are currently using medication that may cause skin sensitivity to light. Doing so may result in photosensitive reaction.
- If unsure whether your medication causes sensitivity to light, confirm with your physician prior to using this device.
- Discontinue use immediately if you experienced any sensitivity to using this device.
- DO NOT USE IF YOU HAVE BEEN DIAGNOSED OR MAY HAVE BLOOD CANCER OR SUN/LIGHT SENSITIVITY.

WARNINGS

- Do not operate device where it could come into contact with water. Doing so could cause an electric shock, result in injury or damage the device.
- Do not use the device if power adapter has fallen into water or is damaged in any way. Doing so could cause electric shock or result in injury. Please contact Customer Service at support@bedrockbioscience.com if power adapter has been damaged in any way.
- Stop using device immediately if you think it is defective. Using a damaged device could result in injury. Contact Customer Service at support@bedrockbioscience.com

- Do not bend or abuse the USB power cord as this can cause electrical shock and problems.
- Use only the supplied power adapter. Use of incorrect power adapter could damage the device. Contact Customer Service at support@bedrockbioscience.com
- Do not attempt to take the device or Lithium battery apart. Any modification or attempted service will void the warranty.
- Discontinue use and contact Customer Service if device overheats or becomes uncomfortably hot to your skin.
- Do not exceed 16 minutes per use. Do not exceed 32 minutes usage per day.
- Avoid direct light contact with testicles, studies show heat may reduce sperm production in men.
- Do not bend or fold the device otherwise it will damage the circuit board. Lay flat for storage and roll gently around penis.

PRECAUTIONS

- Always unplug power adapter from wall outlet when not in use.
- Do not use this device on infants or children without physician approval.
- Do not shine the device lights directly into the eyes.
- This product contains silicone. If you have or suspect you have allergies to silicone, consult your Doctor prior to use.
- Do not use while driving, operating machinery or any other activity requiring concentration and coordination.
- Contact your Doctor if you have any medical questions regarding use of this device.
- Contact your Doctor prior to using if you:
 - Suspect you are taking medications that may cause light sensitivity.
 - Have suspicious or cancerous lesions.
 - Under treatment for ANY cancerous ailment.
 - Have or suspect you have ANY auto immune disease.

DISCLAIMER

The Men's Performance Light intended for relief of symptoms and not intended to diagnose, cure or prevent any disease or condition. Bedrock BioScience™ makes no claims, representations or warranties regarding the ability of this product to cure any physical, skin, or mental condition. All information included or seen regarding Men's Performance Light is for informational purposes only and is not medical advice or claims. A qualified health professional should be consulted with regards to any condition requiring medical attention. Results may vary by user.

MEN'S PERFORMANCE LIGHT

This is a light emitting diode (LED) device that emits infrared energy resulting in temporary increase of blood circulation within the treatment area. The device is designed to be used around the penis as increased circulation has shown positive effects on erectile dysfunction. The device delivers natural light energy in the Infrared and Red spectrum. The device treatment time is controlled by an auto shut off program. There are no user settings or adjustments

DEVICE CHARGING INSTRUCTIONS

1. To charge device, insert USB cable into the Lithium battery on the device.
2. Plug power adapter into electrical outlet.
3. Power indicator light on the rechargeable Lithium battery will turn solid when battery is fully charged (2-4 hours).
4. Power should last 6-8 treatments between charges.

LIGHT SENSITIVITY TEST

Test yourself for light sensitivity prior to initial treatment:

1. Power on your device.
2. Place device on or within 1/4" of the skin on your forearm for three minutes.
3. After 3 minutes check the area. If area has turned red and lasts for more than 2 hours your skin is light sensitive.
4. Discontinue use and contact customer service at support@bedrockbioscience.com

RECOMMENDED USAGE

1. For use for relief from erectile dysfunction and improvement of blood circulation.
2. Use only while alert.
3. Use while in a seated position or laying down.
4. Place the Men's Performance Light with the lighted side of the unit touching the penis. Wrap the sides of the unit around the penis and adhere them together with the magnetic strip. If it is too tight, do not fasten, just wrap around loosely and leave unfastened.
5. One quick press to turn on the device.
6. Device will auto shut off at the completion of the 16 minute treatment.
7. To turn the device off any time during the treatment, press and hold the power button until the LEDs turn off.

FIRST TIME USE:

Turn on the light and allow the light cycle to run for 3 minutes.

If no irritation, perceived extreme heat or over sensitivity or any other non-optimum result occurs, continue for another 4 minutes.

FOR ADDITIONAL USE:

Use the Men's Performance Light one to two times per day. Morning and/or evening usage is recommended. Relief of symptoms may take 4-8 weeks of continuous usage.

MAINTENANCE & CLEANING

Maintenance:

1. This is a medical device. It is important to keep it clean.
2. There are no user-serviceable parts in the device.
3. Device includes only an On/Off button. All settings are set at the factory.
4. Use only supplied power adapter.

Cleaning:

1. Turn device off and ensure it is unplugged from any power source.
2. Clean the device with alcohol wipes or damp cloth.
3. Allow to dry completely before using.

TROUBLESHOOTING

1. System not turning on:

- Is the device fully charged?
- Is power button turned on?
- If device still doesn't turn on, contact Customer Service at support@bedrockbioscience.com

2. Device is not charging:

- Is USB charging cord firmly inserted into device? (Does power indicator light turn on?)
- Try different USB power adapters to ensure it is working properly.
- Try different wall outlets to determine if there is power to the outlet.
- If device still does not charge, contact Customer Service at support@bedrockbioscience.com

3. System shuts off abruptly prior to completion of 16 minute treatment:

- Make sure that the battery is fully charged.
- System is designed to shut off automatically after 16 minutes.

System may shut off automatically after multiple repeated uses if the device exceeds desired temperature. Allow device to cool for several minutes and try again if the device continues to shut off prior to 16 minutes, please contact Customer Service at support@bedrockbioscience.com

TRAVELING WITH DEVICE OUTSIDE USA

The power adapter is designed for USA standard 120V power outlets. When traveling outside the USA, you will need to check the local power requirements and use the appropriate power adapter and/or converter.

CUSTOMER SERVICE

Customer Service can be reached by email at support@bedrockbioscience.com

REPLACEMENT PARTS

There are no replacement parts available for this product at this time.

LIMITED WARRANTY INFORMATION

Congratulations on your purchase. We value you as a client and strive for the highest quality products to earn your satisfaction. Please use this as your reference for any customer service or warranty needs you may have.

WHAT IS COVERED BY THE LIMITED WARRANTY?

This limited warranty is effective only if the device is purchased and used in the United States. This 1 year limited warranty covers defects in materials and workmanship in your Men's Performance Light, if you purchased it from an authorized partner or directly from Bedrock BioScience™. Your 1 year limited warranty is valid for you alone, the original purchaser, and cannot be transferred. If the product was a gift, you can ask the gift giver to send a copy of the receipt to support@bedrockbioscience.com or process the warranty claim on your behalf.

WHAT ISN'T COVERED BY THE LIMITED WARRANTY?

The limited warranty does not cover damage, defects, or failure caused by abuse or misuse, improper care, impact or insertion of foreign objects, use with incorrect power adapter, or loss of device.

HOW LONG DOES THE LIMITED WARRANTY LAST?

The limited warranty begins from the date of purchase and expires 1 year later. The warranty period is not extended if your device is replaced by the company during the warranty period.

WHAT WILL WE DO IF YOUR DEVICE IS DEFECTIVE?

If you discover a covered defect in your Men's Performance Light system before the 1 year warranty period expires, contact Customer Service at support@bedrockbioscience.com for assistance. All limited warranty services must be authorized and performed by Bedrock BioScience™.

WARRANTY LIMITATIONS & DISCLAIMERS

This warranty gives you specific legal rights and is in addition to your statutory rights. Your remedies for the breach of this limited warranty are limited solely to the repair and replacement of your device as set forth in this warranty statement. To the maximum extent permitted by applicable law, Bedrock BioScience™ disclaims all implied warranties. If applicable law does not permit the disclaimer of implied warranties, then the duration of implied warranties are hereby expressly limited to the duration of the express limited warranty set forth above. To the maximum extent permitted by applicable law, in no event shall Bedrock BioScience™ be liable for any special, incidental, consequential, punitive, or indirect damages based upon breach of warranty, breach of contract, negligence, strict product liability, tort, or other legal theory, even if Bedrock BioScience™ is advised or aware that such damages might arise. Such damages include, but are not limited to, loss of device, cost of any substitute equipment, the claims of third parties, or damage to property. If the applicable law does not allow limits on warranties, or on remedies for breach in certain transactions, the limits in this paragraph and the disclaimer or exceptions from warranty may not apply. This limitation does not apply to claims for death or personal injury caused by a defect in the device or any other liability that cannot be excluded or limited by law. These warranty terms, conditions, and limitations allocate the risks of product failure between Bedrock BioScience™ and you, the purchaser of Men's Performance Light system. This allocation is recognized by both parties and is reflected in the price of Men's Performance Light system. By using the Men's Performance Light you acknowledge that you have read these terms, conditions, and limitations, understand them and are bound by them.

Distributed by:

B E D R  C K™

BIOSCIENCE

Irvine, CA 92614

For Questions Email:

info@bedrockbioscience.com

Made in China